# BRAEBON

*For Today’s Business of Sleep™*

**Canada:**  
1-100 Schneider Road  
Kanata, ON, Canada, K2K 1Y2  
01.613.831.6690  
Toll-free (N.A.): 1.888.462.4841

  CEpartner4U BV  
Esdornoiaan 13  
3951 DB Mearn  
The Netherlands  
Tel: +31.343.442.524

**en**  **Body Position User Guide**  **All Models**

#### Intended Use

The BRAEBON MEDICAL CORPORATION *Ultima* Body Position Sensor™ is intended for use during sleep disorder studies as an indicator of body position. The sensor uses a three-volt lithium battery and outputs either variable voltage or variable resistance directly into a DC amplifier, a DC capable headbox, multiplexer, or the MediPalm®.

**Warning:** Body position sensors are for diagnostic purposes only and are not intended as apnea (apnoea) monitors.

**Caution:** US Federal law restricts this device to sale by or on the order of a physician. Do not wash the body position sensor. Do not steam autoclave the Body Position sensor and never immerse the sensor in liquids. Use only isopropyl alcohol pads to clean the sensor.

**Note:** Battery life expectancy is approximately 2 years and the LED will indicate impending battery failure at approximately 22 months. Refer to www.braebon.com for more information. Product specifications are subject to change without notice.

#### How to Use

Refer to your sleep system user guide for manufacturer’s recommended settings. Red 1.5 mm touchproof connector is positive; black is negative. Use either a direct coupled (DC) amplifier or connect directly to the recorder’s multiplexer.

The Body position sensor is always on and will last approximately two years. With the up arrow pointing to the patient’s head, carefully wrap the Velcro adhesive found on the back of the body position sensor around the thoracic belt. For the body position sensor to operate properly, the sensor must be flat and centered on the chest immediately above or below the nipple line. If the sensor is 25 degrees above or below the horizontal plane while lying supine or prone, it will indicate the standing position.

After positioning the belt, gently press the sensor to engage the Velcro backing. This ensures that the sensor remains stationary.

### Table 1: Signal Output

Body Position	Voltage ± 0.1
right side	0.50 volts
left side	1.00 volts
supine	0.25 volts
prone	0.75 volts
upright	0.00 volts

#### Specifications

Operating Temperature: 5°C (41°F)-32°C (90°F)

Storage Temperature: 0°C (32°F)-60°C (140°F)

Operating/Storage Humidity: 10–95% Non-condensing

#### Disinfection

**Caution:** Do not steam autoclave the body position sensor or immerse the sensor in any liquids. Use only isopropyl alcohol pads to clean the sensor.

For additional information, refer to the APIC guidelines for selection and use of disinfectants (*American Journal of Infection Control*, Vol. 18, number 2, April, 1990, p. 99-117).

#### Warranty

BRAEBON MEDICAL CORPORATION warrants to the first consumer that the Ultima Body Position Sensor™ ( the “Sensor”), when shipped in its original container, will be free from defective workmanship, performance and materials and agrees that it will, at its option, either repair the defect or replace the defective Sensor or part thereof at no charge to the purchaser for parts or labor for a time period of one year from the date of purchase. The warranty described herein shall be the sole and exclusive warranty granted by BRAEBON MEDICAL CORPORATION and shall be the sole and exclusive remedy available to the purchaser. Use of the Sensor constitutes total and complete acceptance of this warranty. Correction of defects, in the manner and for the time period described herein, shall constitute complete fulfillment of all liabilities and responsibilities of BRAEBON MEDICAL CORPORATION to the purchaser with respect to the Sensor and shall constitute full satisfaction of all claims, whether based on contract, negligence, strict liability or otherwise. In no event shall BRAEBON MEDICAL CORPORATION be liable, or in any way responsible, for any loss of revenues or damage, direct, incidental, or consequential, including property damage, loss of profit, or personal injury resulting

from the use or misuse of, or the inability to use this product. Nor shall BRAEBON MEDICAL CORPORATION be liable, or in any way responsible, for any damages or defects in the Sensor which were caused by abuse, misuse, tampering, neglect, incorrect battery type, or repairs or attempted repairs performed by anyone other than an authorized service provider.

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